




Manual for 7 in 1 Hydro Dermabrasion

Operating methods and procedures


1. Hydrodermabrasion: facial cleansing, peeling, pore cleansing



Select a suitable water mill according to the skin and rotate the knob in front of the screen to the bottles A, B or D that need to be used.

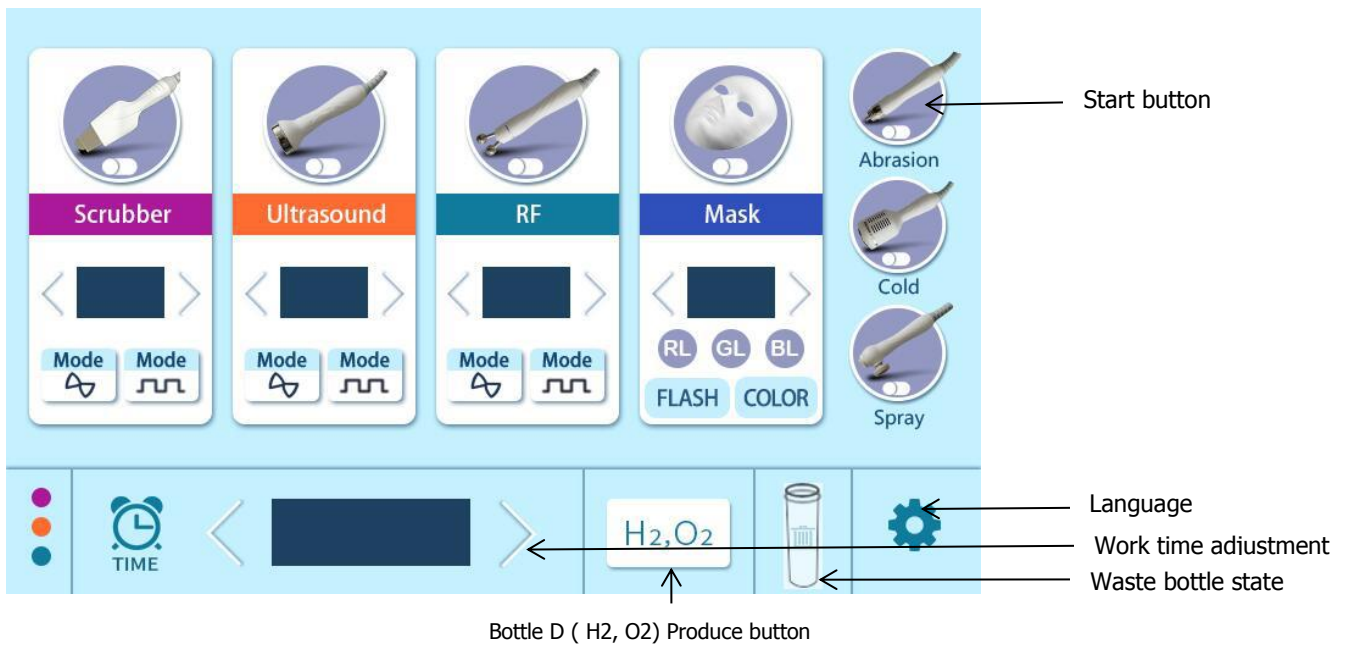


Adjust negative pressure suction and water flow to clean the whole face.

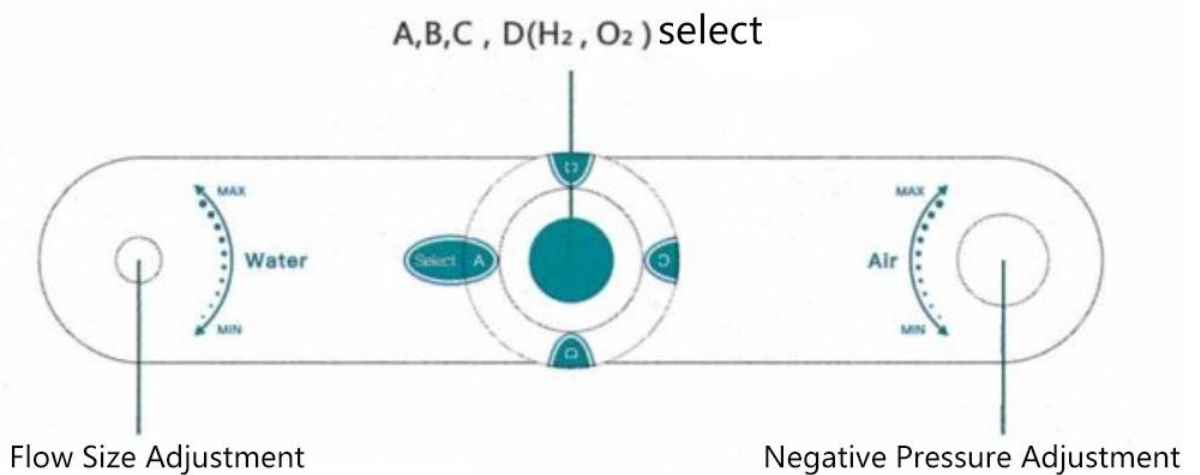

2.oxygen spray gun : skin rejuvenation




Pour the diluted essence into the container on the spray pen.

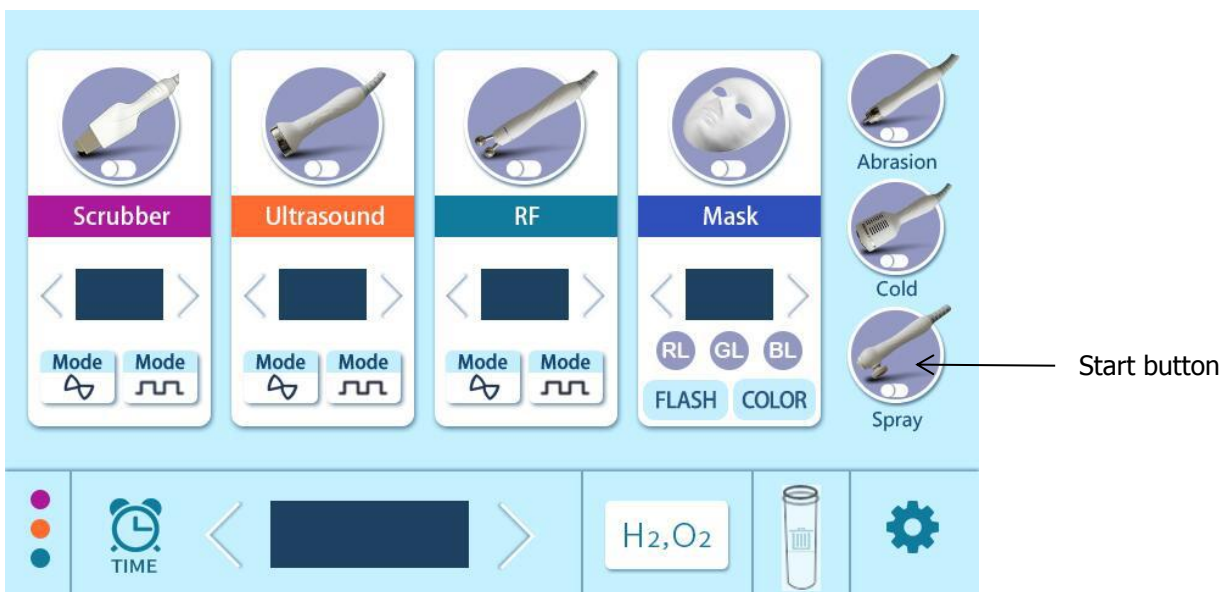

3 cold hammer : cold down skin after treatment.




Operate the front contact area of the handle

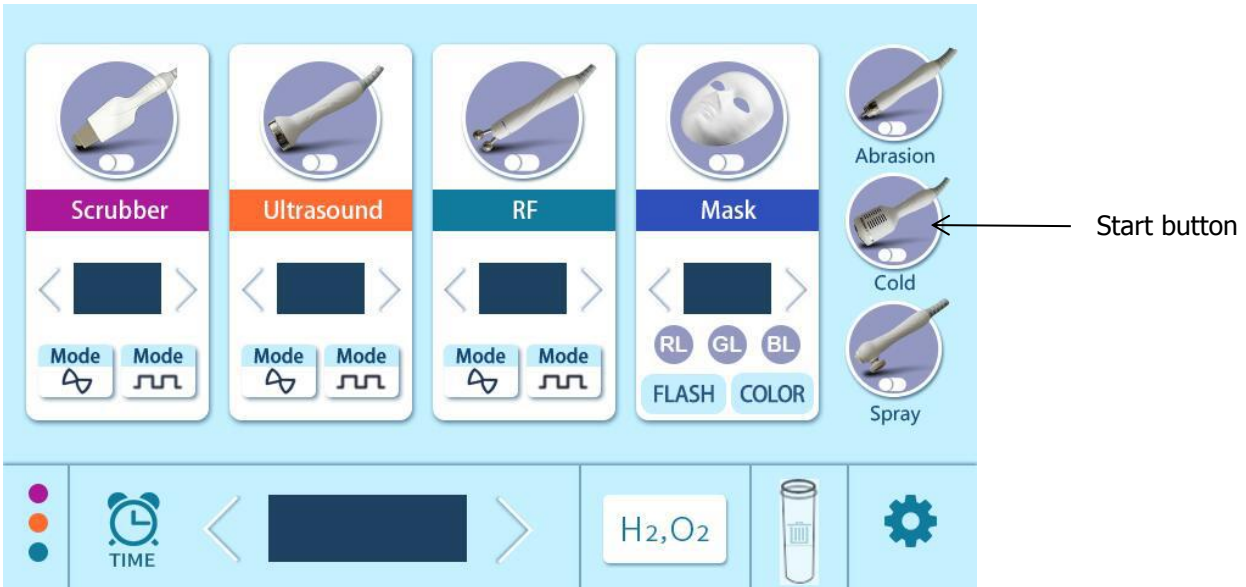

4. Multipolar RF: compact shaping, face lifting




Apply proper amount of nutritious gel on the skin. Close the RF head to the action site. Operation time is 10-15 minutes.

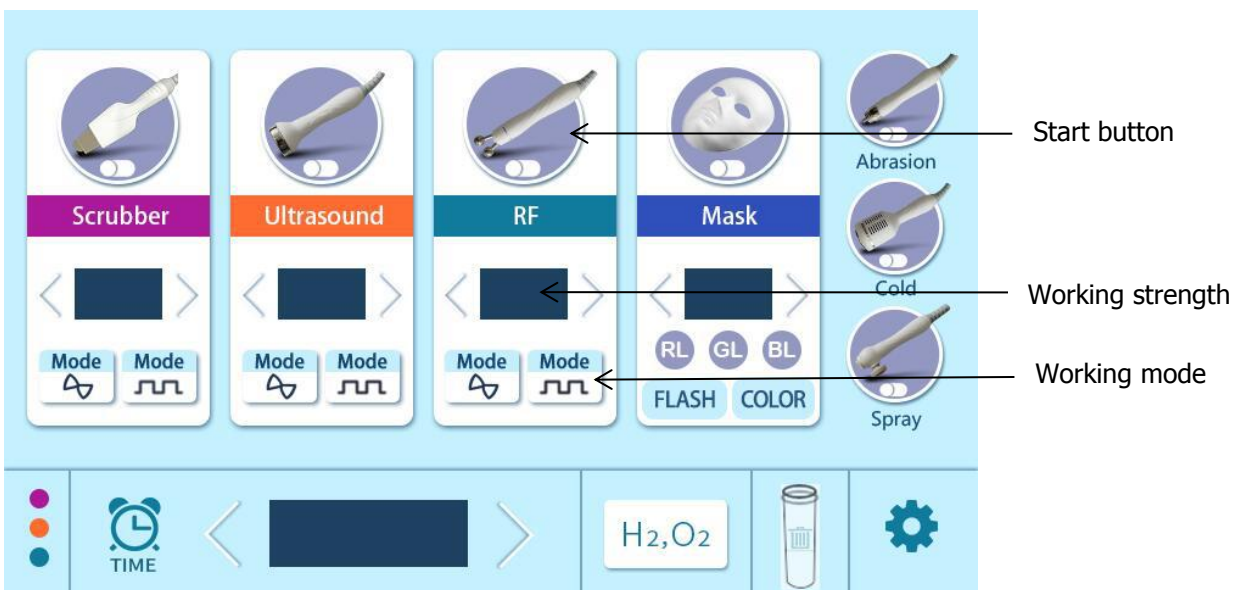

5. Ultrasonic head: promote microcirculation, skin rejuvenating



Apply proper amount of essence to your skin. Tighten the acoustic head to the action area. Operation time is 5-8 minutes.

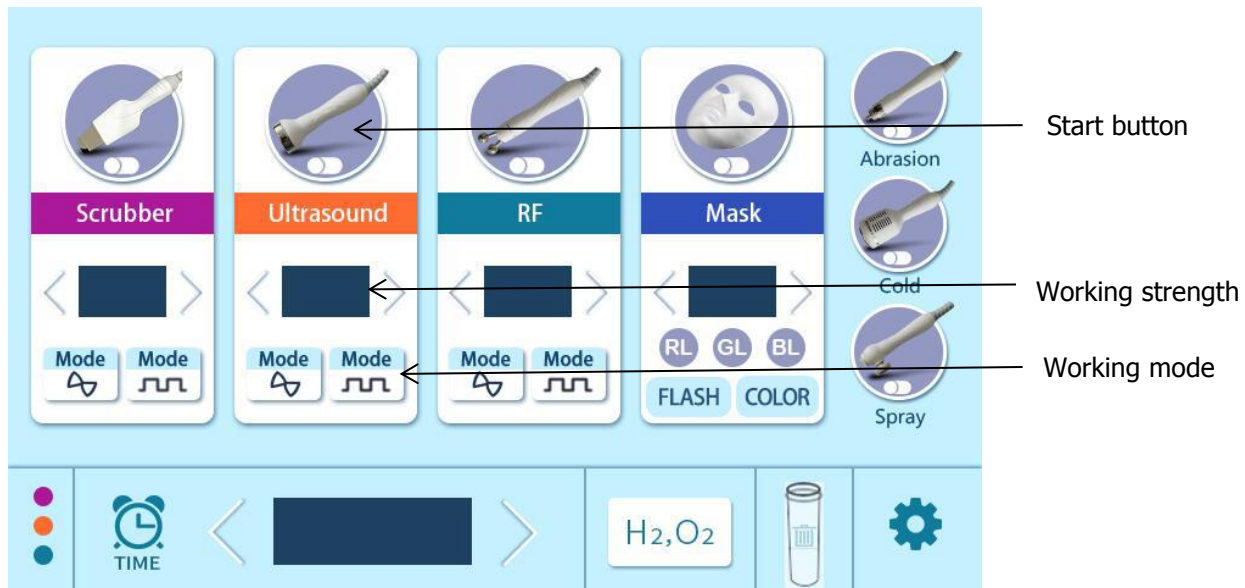


6. Skin Scrubber: let jaw line tightened, Bio raise the overall tightening of facial skin



After cleaning the skin, The probe is inclined to the skin at 45 degrees and moves from bottom to top with moderate speed. Operation time is 10-15 minutes.


Start button

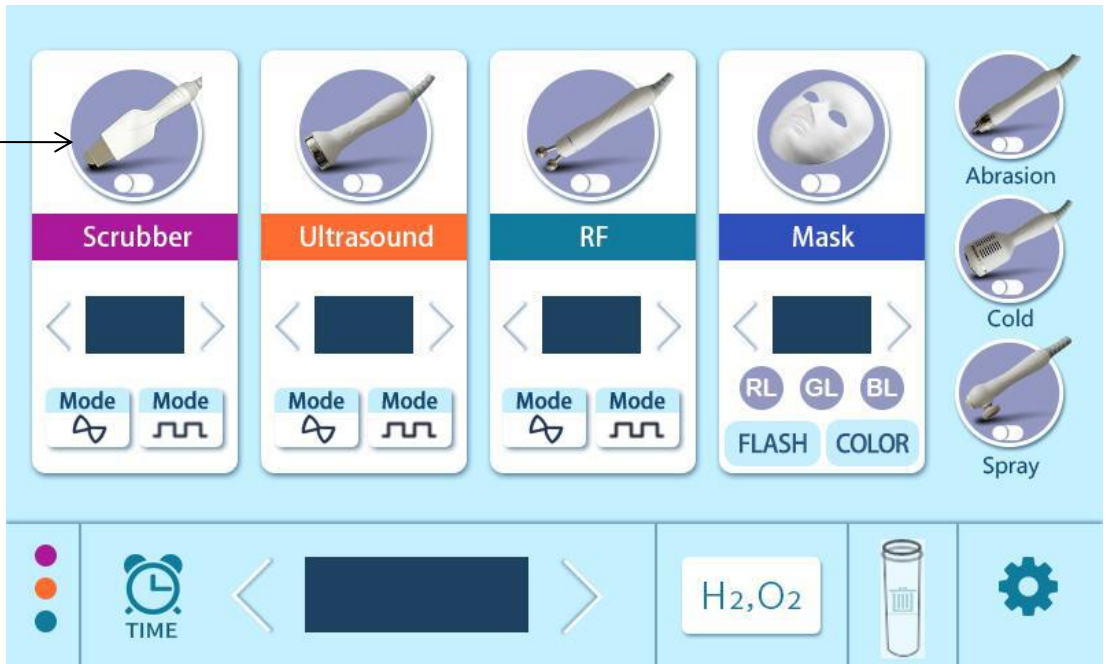

7. Facial Mask



Apply on the face after the treatment, it has 5 modes of color to select.

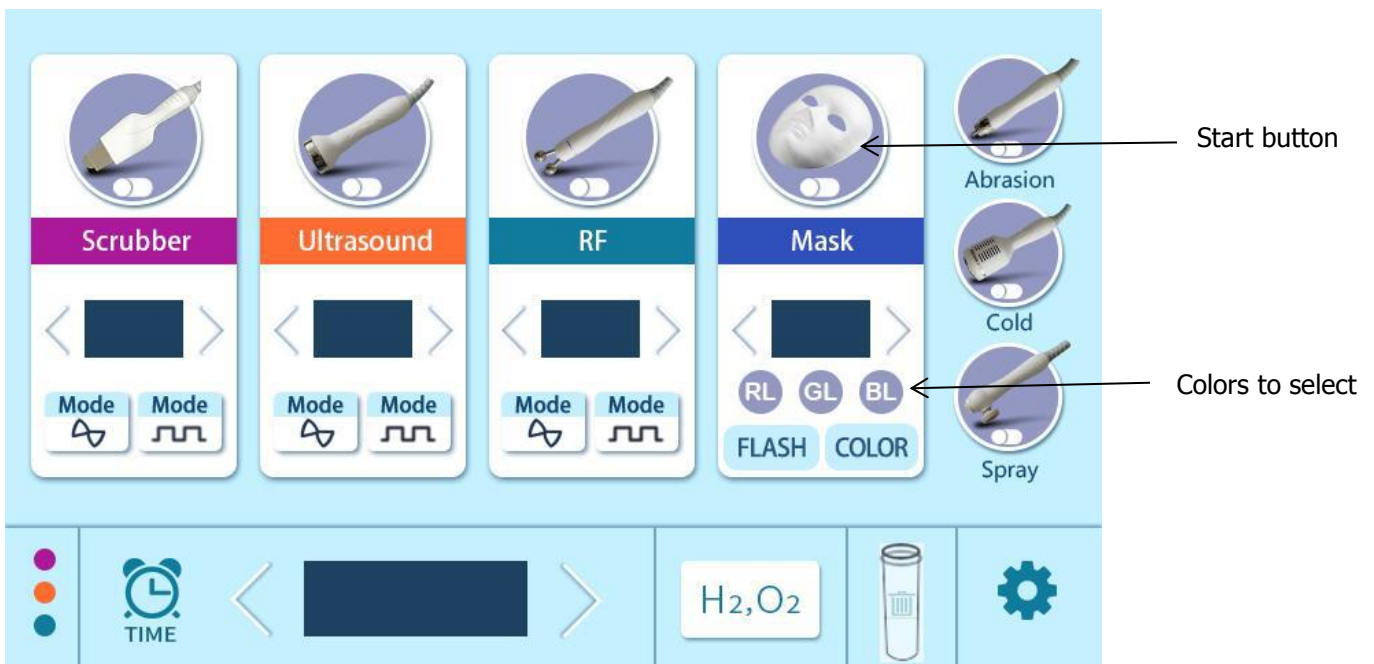

Advantage

- 1: Hydro-dermabrasion, applicable to regular or sensitive skin, Or skin with whitehead, comedo, acne, etc.
- 2: Cleaning&washing: deep cleaning, clear skin statures, minimally invasive scars, and clearing blackheads, remove deep skin dirt
- 3: Effective and direct moistening: supply sufficient water molecules to skin while cleaning
- 4: Used the skin care products to achieve a large variety of treatment aims like wrinkle/pigmentation removal, skin lightening and whitening