



User Manual

Warning

Dear User

Thanks for using our company's products, in order to fully make use of the products, We sincerely suggests follow:

1. please read the instruction in detail and keep it well for further reading and consulting.
2. Please follow the instruction to fit and operate and machine correctly.
3. Please dont remove and change any accessories of the machine.
4. It forbids others to open and remove the mainframe except the authorized people in the company.
5. It forbids others to change and maintain the machine except the professional people
6. Please contact us if there is any trouble during operation.

1.Theory

High Intensity Focused Ultrasound (HIFU) directly delivers heat energy to skin and subcutaneous tissue that can stimulate and renew the skin's collagen and thus consequently improving the texture and reducing sagging of the skin. It literally achieves the results of a facelift or a body lifts without any invasive surgery or injections, moreover, an added bonus of this procedure is that there is no downtime. and also, it works equally well for people of all skin colours, in contrast to that of lasers and the intense pulse lights.

It's heating directly to deeper tissue, in order to lift the skin. It is performed in the office with only ultrasound gel applied to the skin. An ultrasound screen image allows the physician to visualize the level of treatment before energy is applied to the targeted tissue. The treatment takes from 45 to 90 minutes depending on the area(s) treated.

Ultherapy is a safe and effective non-invasive ultrasound (US) treatment that can lift and tighten facial skin and muscle without downtime. It is targeted for Lifting skin on the neck, under the chin, on the brow etc.

2.Advantages

- More than 10,000 shots according to quality inspection and clinical tests, ensuring that each shot effects into the right depth set by default.
- Equipped 5 heads (1.5mm, 1.5mm, 3.0mm, 4.5mm, 13mm), precisely acting on different depths of the skin. The energy will skip the epidermis, leaving no damages.
- It will produce thermal effect to the dermal collagen fibers and stimulate the fat layer and SMAS. The result is far better than Thermage.
- The skin will be lifted and tightened at once, and effect will last for 12-24 months.
- There are no post-surgery down time and you can resume your normal activities immediately
- Easy to control and operate, no consumables, cost-saving.

3.Application:

1. Remove wrinkles on around forehead, eyes, mouth, etc.
2. Lifting and tightening both cheeks skin and body slimming
3. Improving skin elasticity and shaping contour.
4. Improving jaw line, reducing "marionette lines"
5. Tightening the skin tissue on forehead, lifting the eyebrows lines.
6. Improving skin complexion, making the skin delicate and bright
7. Match with injecting beauty like Hyaluronic acid, collagen, to resolve more aging

problem

8.Removing neck wrinkles, protecting neck aging.

4.Why HiFu treatment is better than other treatment?



Two key points in skin tightening treatment: From shallow to deep facial structure are divided into:Epidermis,dermis,subcutaneous fat,SMAS,muscles and bones.Traditional surgery face lift surgery is to suspend SMAS,achieving skin tightening and lift.Now using HIFU non-instructive methods can easily reach unprecedented treatment depth.

HIFU ultherapy depth is 1.5mm-13mm.Besides acting on fibrous SEPTA,ultherapy Can also act on deeper layer SMAS,the key point of traditional face lift surgery. Therefore, HIFU has broken the previous limit, is the first instrument for non-intrusive SMAS treatment. Every single HIFU energy point(TCP) under the skin can make the temperature reach 65-70C,is the strongest temperature of all non-invasive skin tightening device.

**No surgery, No bleeding, No scars,
No trauma, No radiation, No side effects**

5.Function&Accessories:



DS4-3.0MM



DS4-1.5MM



ES4-1.5MM



Transducer handle

DS4-4.5MM



DS4-13MM



1. For the transfer of frequency 4Mhz high intensity focused ultrasonic to the skin and subcutaneous tissue, ultrasonic penetrating the dermis depth of 4.5mm, direct access to the SMAS layer of subcutaneous tissue, forming “coagulation” area, treat thick skin, such as cheeks.

2. for the transfer of frequency 7Mhz high intensity focused ultrasonic to dermis skin tissue, ultrasonic penetrating the dermis depth of 3.0mm, specific responsibility to activate collagen of the dermis skin, effectively consolidate the outline lifting effect, also can improve coarse pores and reducing wrinkles.


3. for the transfer of frequency 7Mhz high intensity focused ultrasonic to the tissue of the epidermis, ultrasonic penetrating the depth of 1.5mm, specific responsibility to activate the thin epidermis tissue.

Frequency	4MHZ	4MHZ	7MHZ	7MHZ	10MHZ
Depth	1.5MM	1.5MM	3.0MM	4.5MM	13MM
Treatment scope	1. Acne Removal 2. Coarse Pore 3. Forehead Wrinkles 4. Skin Restructuring	1. acupuncture remove 2. Remove crow's feet	1. Face lifting 2. Nasolabial folds 3. Wrinkle Removal	1. Face lift	1. Body slimming


6.operation steps:

- 1.Clean the Skin.
- 2.Take a photo before treatment , and paint on the preoperative lifting location with make - up pencil .
- 3.According to the skin thickness to select appropriated transducer .
- 4.Set suitable length , distance , energy ; we suggestion that the length adjust 25mm , distance adjust 2mm , energy generally start from 0.1J (according to customers need)
- 5.Applying ultrasonic plentiful gel as couplant between transducer and skin.
- 6.face lifting operate on the note of power line, forming a line thermal condensed node in the specified depth under skin when the transmitting button is pressed per time,operating continued on the powder line until the specified location transmission is completed. The treatment parts such as face, chin and both sides of neck.
- 7.don't operate on marginal mandibular nerve and supra or bital nerve.
- 8.no recovery period after treatment,daub some moisture product and make up immediately.
- 9.only once, the comprehensive effect will gradually appear in two or three moths, Some people may do this treatments several times.


7.screen:

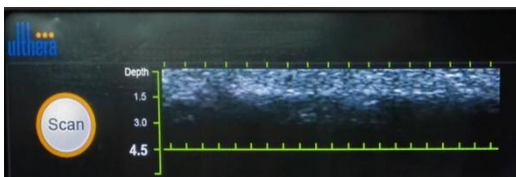


treatment depth display

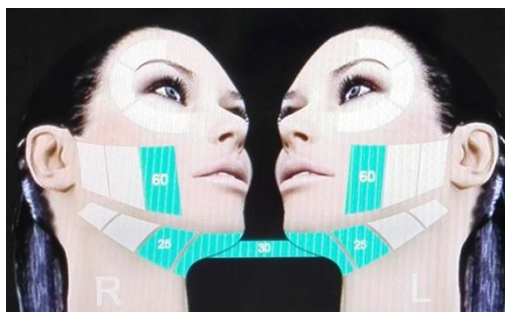


displaying which area is being treated

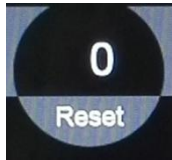


Shot count

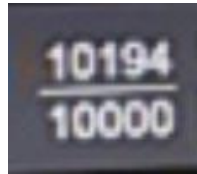

total shots

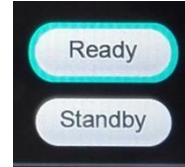

ready/standby

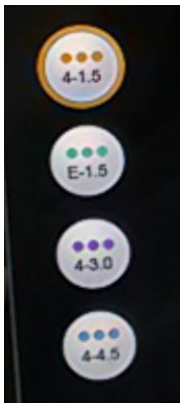


displaying which head is in use

1.5mm for eye,forehead


3.0mm for face

4.5mm for neck

13mm for body

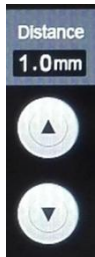

distance

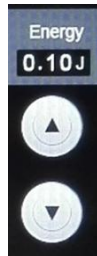

energy



length



- 1 machine light
(when press ready key , this light on)
- 2 working light
(when press control key , this light on)
- 3 control key
(when press control key , machine working)

8.Parameters:

Model Name	UIthera Skin Lifting Beauty Equipment
Technology	HIFU (High Intensity Focused Ultrasound)
Power	150W
Voltage	110-240V, 50/60Hz
shots	10000/ each head
Number of heads	5
Energy type	HIFU 4Mhz/7Mhz/10mhz
distance	1.0- 4.0 mm (0.2 mm step)
length	5 - 25 mm (1.0 mm step)
Energy output	0.1 - 2.5J/cm2(0.1J step)
Depth	1.5mm/1.5mm / 3mm / 4.5mm/13mm
Screen size	15 color touch screen

9.cautions after face lifting:

- 1.strengthen the miniaturizing and sun scream.
- 2.Don't take any medicine and drink alcohol before and after 24hours, follow the nursing advice.
- 3.don't soak hot springs and have a steam bath.
- 4.It is normal that the skin feel fry after face lifting, pay attention to water replenishing.suggestions for applying mask at least three times a week.

10.Adapt to the crowd:

- 1.people who have facial skin sagging,wrinkles,skin aging and rough,reshape the face curve, effectively improve the eye region and neck tissue sag, posits.
- 2.Tightening the loose skin, removing real and false wrinkles,repair stretch marks.
- 3.Acnes
- 4.Systemic anti-aging(neck and back tightening and shaping, breast shape adjustment, waist and abdomen shaping and modifying,shape the buttocks and legs.

5.Suitable for a variety of wrinkles and deep wrinkles, forehead wrinkles, eye wrinkles,purify end, nasolabial fold, lip wrinkles, frown lines etc....

11.Not meet the crowd:


People who implant a pacemaker or other similar electronic apparatus and implant other beauty products which it is dissolved by temperature.

12.compare effect:

Face effect



Body Effect

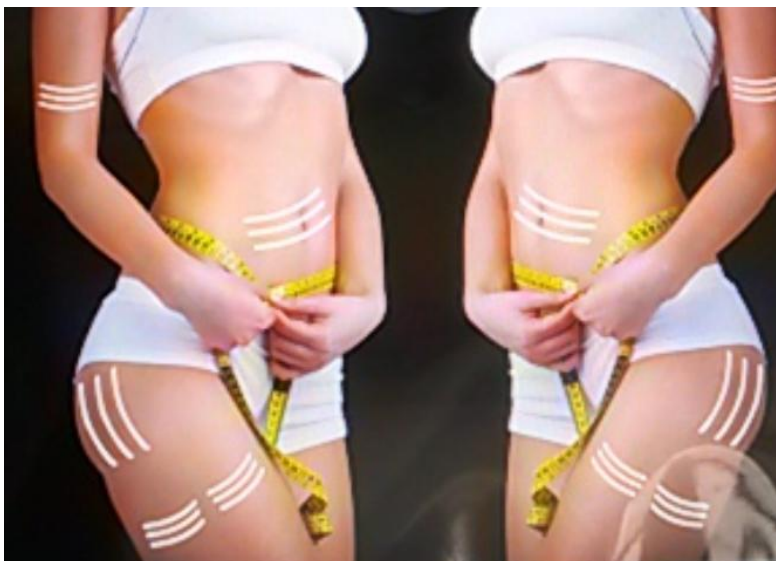

14. Notes:

- 1, ultrasonic knife operation after a temporary slight swelling phenomenon is very normal. Note 1 week don't go hot sauna, yoga and other high-temperature environment after the operation, and is not recommended for exposure.
- 2, before the operation to maintain the skin, hair part clean, use the day cannot make-up, the current traces of make-up to clean. Please does not use hot water wash your face
- 3, three days (not more than the temperature of the water can be); note that replenishment, suggest a week at least apply mask 3 times.
- 4, a week does not eat soy sauce pigment heavy things, alcohol, spicy food.

15. unsuitable for

1. implanted cardiac pacemaker or other similar electronic apparatus.
2. Pregnant women.
3. Over the past 9 months taking oral isotretinoin therapy were (Accutane / Roaccutane / Oratane).
4. Internal filler, if the face with gold implantation, it is not recommended to do.
5. Serious heart disease, diabetes, hyperthyroidism patients.
6. Operation wound not healed.
7. Patients with advanced cancer. The effect of unrealistic fantasies.

16. Equipment Maintenance:

1. gently, does not put the instrument placed in direct sunlight
2. regularly with a dry towel clean surface, machine parts surface dust and dirt.

3.before and after each use will probe cleaning and disinfection (wipe the surface with a 75% medical alcohol, prohibit the use of alcohol)

4.to prevent cross infection, each operation probe for personal use!

17.Packing List

Main Device	1pcs
Frame	1pcs
transducer handle	1pcs
Treatment head	5pcs(1.5mm,1.5mm,3.0mm,4.5mm,13mm)
USB	1pcs
Fuse	2pcs
Power cord	1pcs
User manual	1book

18. Maintenance of Machine

I 【precautions】

- 1) No stress on the touch - LCD, prevent it from breaking up;
- 2) Keep the handle clean. It's not allowed to clean it with ethanol;
- 3) The handle is fragile, please do be careful when you pick up the headpiece;

19. After-sale Service And Maintenance

The term of maintaining guarantee is main device 1 year and spare part 3 months from the date of purchasing according to the receipt. (Expendable is not included in it.) Others not included in maintaining guarantee or over the time, we also supply to maintain and renew but need charges. Please contact with the dealer if need change of purchasing the parts and easily expendable products.

- A.** We will guarantee the quality of our equipment, normally there is no quality problem
1. If there is a deficiency, please email or call us on the reasons, it would be best to deal with by email or telephone. Because some of the problems are caused by using the machine in a wrong way.
 2. We suggest the client to take photos of the area where the problem occurs, we will pass them to our technician , our technician will give a satisfactory answer within 1-7 working days
 3. If the problem comes from the spare part within the warranty, we could consider in changing the spare part to our client, which could be sent with his/her next order. If the next order is uncertain, we could pay freight cost to send the spare parts.
 4. If the problem comes from the main device within the warranty, we could advice our client to post the device back to our company for repair by ordinary post, or we could re-send a new one to our client. If the cost of posting the problematic device back to BEST is more than the value of a new device, BEST would take it into consideration, according to the actual status; apply to re-send a completely new device back to our client.
- B.** We won't supply maintaining guarantee if troubles as below:
- a) Disassembling the machine by self;
 - b) Fall it without caution;
 - c) Lack of reasonable keeping and protect;
 - d) Not operate according to the user manual.
 - e) Expendable reasons

If changes of the design and specification of the machine, we will not notice you in addition. If any queries, please feel free to contact with the local marketing company.

All in one, we will try our best to serve you and also need your fully cooperation. Thanks.

Product name :			Purchase date:			deadline:		
Seller:			Add.:			Tel.:		
Customer:			Add.:			Tel.:		
Repair date			Reason of the fault			Repair content		
Maintain list								