

Diode laser Clinical Parameters Setting Reference

Face				
Skin Type	Frequency	Energy(J/cm ²)	Pulse Width (Ms)	Treatment time
1	Automatic	16	Automatic	6~8 times treatment/21~45 days interval between two treatments
2	Automatic	14	Automatic	
3	Automatic	11	Automatic	
4	Automatic	6	Automatic	
5	Automatic	3	Automatic	
6	Automatic	2	Automatic	

Armpit				
Skin Type	Frequency	Energy(J/cm ²)	Pulse Width (Ms)	Treatment time
1	Automatic	15	Automatic	6~8 times treatment/21~45 days interval between two treatments
2	Automatic	14	Automatic	
3	Automatic	12	Automatic	
4	Automatic	5	Automatic	
5	Automatic	4	Automatic	
6	Automatic	3	Automatic	

Body				
Skin Type	Frequency	Energy(J/cm ²)	Pulse Width (Ms)	Treatment time
1	Automatic	16	Automatic	6~8 times treatment/21~45 days interval between two treatments
2	Automatic	14	Automatic	
3	Automatic	10	Automatic	
4	Automatic	5	Automatic	
5	Automatic	4	Automatic	
6	Automatic	3	Automatic	

Arm				
Skin Type	Frequency	Energy(J/cm ²)	Pulse Width (Ms)	Treatment time
1	Automatic	18	Automatic	6~8 times treatment/21~45 days interval between two treatments
2	Automatic	13	Automatic	
3	Automatic	10	Automatic	
4	Automatic	5	Automatic	
5	Automatic	4	Automatic	
6	Automatic	3	Automatic	

Leg				
Skin Type	Frequency	Energy(J/cm ²)	Pulse Width (Ms)	Treatment time
1	Automatic	15	Automatic	6~8 times treatment/21~45 days interval between two treatments
2	Automatic	12	Automatic	
3	Automatic	8	Automatic	
4	Automatic	5	Automatic	
5	Automatic	4	Automatic	
6	Automatic	3	Automatic	

Bikini				
Skin Type	Frequency	Energy(J/cm ²)	Pulse Width (Ms)	Treatment time
1	Automatic	17	Automatic	6~8 times treatment/21~45 days interval between two treatments
2	Automatic	14	Automatic	
3	Automatic	9	Automatic	
4	Automatic	5	Automatic	
5	Automatic	3	Automatic	
6	Automatic	2	Automatic	

